






**Cosmedico**  
**E 2000/220V L**  
**Metal halide lamp for**  
**sun tanning equipment**

Typical Relative Spectral Radiant Flux

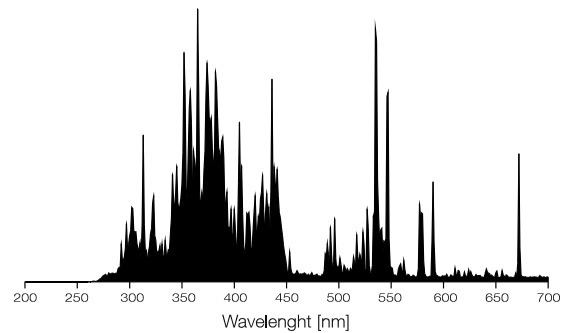

Article-No.	24203
FDA-ACC. No:	9220059
Total length (A)	171 mm
Lighted length (B)	75 mm
Bulb diameter (C)	Ø 28 mm
Ceramic base	Ø 10 mm
Leads (Li)	370 mm
Bulb (quartz glass)	FDA
Lamp voltage $U_{Lp}$ .	145 V
Current $I_{Lp}$ .	15 A
Lamp wattage $P_{Lp}$ .	2000 VA
Supply voltage	230 V 50 Hz
Starting voltage	3-4,5 kV
Intial phase	3 min.

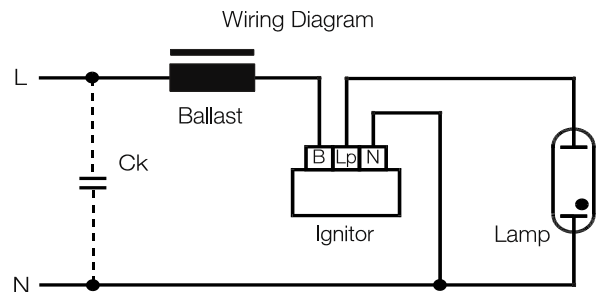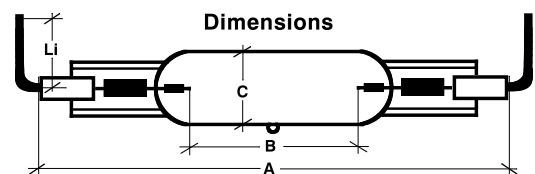

We recommended a parallel connection from two ballasts Cosmpower S 1000W (#74423/50Hz) and a parallel connection from two ignitor Cosmpower S Z 1000 (#71813). The useful life is determined on the operation condition of the lamp (for example type of ballast / ignitor used, on / off cycle, cooling conditions, etc.). We recommend to change the lamps if the reduction of intensity is down to 75% of its original output.

To avoid failure of the lamps they have to be operated in horizontal position (+/- 30°) with adequate cooling. If the following operating temperatures are exceeded, the lamp may be destroyed:

bulb temperature  $\leq 950\text{ }^{\circ}\text{C}$  and  $\geq 750\text{ }^{\circ}\text{C}$   
 seal temperature  $\leq 350\text{ }^{\circ}\text{C}$

Remove contaminations (e.g. finger-prints) with 100% alcohol!

**Please note!**

Follow sunlamp equipment manufactures instructions. Tubular metal halide lamps emit a very intensive UV-radiation. Filter and safety glasses must be used as eyes and skin may be damaged within seconds otherwise.